

Packing Guide for UN3373 Biological Substances Category B by Air using International Shipper Boxes

THIS BOX IS TO BE USED ONLY FOR INTERNATIONAL SHIPMENTS (AMBIENT ONLY)

Step 1

Ensure primary containers are sealed and are clean.

Step 2

Insert the primary container inside of the 95KPA transport bag with the absorbent material.

Step 3

Remove the white strip and seal the bag by folding along the black lines.

Step 4

Make up the outer box as directed and place the sealed bag and documentation inside the box.

Step 5

Close the box and place your address label with full contact details of the delivery address on the front.

Step 6

Complete Eurofins Lablink delivery docket and attach to box. The package is now ready for collection.

